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EN
 Warning and safety instructions
 Observe the replacement intervals for the
 protective mask in accordance with the
 occupational safety guidelines for your
 work area. The mask must be replaced
 immediately if it is damaged or if
 breathing resistance increases. Do not wear
 the mask for longer than one shift.
 Facial hair can affect adequate fit.
 The mask must not be used in areas with
 a low oxygen content (less than 19.5%)
 or where there is an explosion risk, for
 work involving asbestos, paint spraying, or
 sandblasting, if you taste or smell toxins,
 for protection from gases or vapours, if the
 concentrations of toxins are unknown or
 are an immediate danger to life or
 health.

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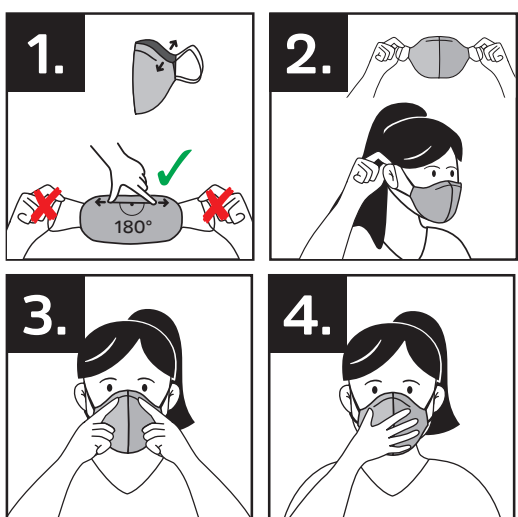
EN
How to use
 1. Unfold the mask only by bending the nose clip, not by pulling the ear loops. Please make sure that the nose clip is fully bent open (at a 180° angle). Check for damage and contamination before use.
 2. Hold the mask by the ear loops with both hands. Place your chin in the mask first, then pull the mask over your nose. The nose clip should be positioned just above the nose bridge. Pull ear loops behind your ears.
 3. Adjust the nose clip with both index fingers from the center outwards.
 4. Check correct fit of mask. The Mask should fit tightly and without gaps onto the face. Hold both your hands in front of the mask and exhale forcefully. If air comes out of the nose area, adjust the nose clip to suit, or press it down more.

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EN Information booklet

Particle-filtering half mask EN 149:2001 + A1:2009 FFP2 NR

Filtration of at least 94 % of pollutants/particles from the air, up to a size of 0.6 µm

Limits of use: protection against harmful or low-toxicity and solid, aqueous and non-volatile liquid particles and aerosols, at up to 10 times the individual threshold value in accordance with DIN EN 149:2009-08

CE certification 2016/425/EU FORCE Certification A/S 2605 Brøndby (Code Nr. 0200)

Made in Germany (Vogtland, Saxony)

The full technical data sheet is available in PDF format in the download area of the TechniSat homepage at www.technimask.de. The complete text of the EU Declaration of Conformity is available at the following web address: <http://konf.tsat.de/?ID=23083>

Specification

FFP2 respirator face mask

Part. no.: 0000/2831

Storage life (in original packaging): 5 years in the sealed bag.

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Certyfikat CE 2016/425/EU FORCE Certification A/S 2605 Brøndby (Code Nr. 0200)